

Gridlock Ankle Plating System

Instructions for Use

900-01-011 Rev H
May 26, 2017

Description

The Gridlock Ankle Plating System is comprised of a variety of contoured titanium plates with different shapes and sizes designed for internal fixation. The Gridlock Ankle Plating System utilizes solid core threaded standard and locking screws, cannulated screws, and syndesmotic screws in various diameters and lengths. Available plates, screws and instrumentation are packaged as a single system. System instrumentation includes drill bits, countersinks, guide wires, depth gauges, bone clamps, bending instruments, drill guides, drill sleeves, a screw removal tool, driver shafts, handles, and ancillary instruments to facilitate the placement of the plates. The plates, screws, and guide wires are intended for single use only.

Implant Materials

All Gridlock Ankle Plating System plates are made from commercially pure Titanium (ASTM F-67). All Gridlock Ankle Plating System screws are made from Titanium Alloy (ASTM F-136). Associated instrumentation is made from medical grades of titanium, stainless steel, anodized aluminum, and plastic.

Indications

The Gridlock Ankle Plating System is intended for use in trauma and reconstructive procedures of the small bones in the hand/foot, ankle, distal tibia, fibula, and other bones appropriate for the size of the device.

Contraindications

Use of the Gridlock Ankle Plating System is contraindicated in cases of active or suspected infection or in patients who are immunocompromised; in patients previously sensitized to titanium; or in patients with certain metabolic diseases. It is further contraindicated in patients exhibiting disorders that would cause the patient to ignore the limitations of internal fixation.







Warnings

1. Re-operation to remove or replace implants may be required at any time due to medical reasons or device failure. If corrective action is not taken, complications may occur.
2. Use of an undersized implant in areas of high functional stresses may lead to implant fracture and failure.
3. Instruments, guide wires, olive wires, and screws are to be treated as sharps.
4. Re-use of devices indicated as single use can result in decreased mechanical and clinical performance of devices.
5. The washers are not intended for use with the plates; washers are intended for use with non-locking screws only.

Maintaining Device Effectiveness

1. The surgeon should have specific training, experience, and thorough familiarity with the use of plating systems.
2. The surgeon must exercise reasonable judgment when deciding which plate and screw type to use for specific indications.
3. The Gridlock Ankle Plating System is not intended to endure excessive abnormal functional stresses.
4. The Gridlock Ankle Plating System is intended for temporary fixation only until osteogenesis occurs.
5. All Gridlock Ankle Plating System implants and instrumentation may be required for each surgery. Failure to use dedicated, unique Trilliant Surgical instruments for every step of the implantation technique may compromise the integrity of the implanted device, leading to premature device failure and subsequent patient injury. Failed devices may require re-operation and removal.
6. Carefully inspect the implants prior to use and inspect the instruments before and after each procedure to assure they are in proper operating condition. Instruments that are faulty, damaged or suspect should not be used.
7. The Gridlock Ankle Plating System should be used in a sterile environment.

Instructions for use, Gridlock Ankle Plating System

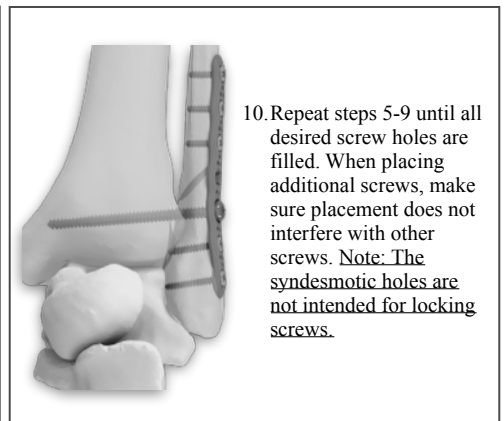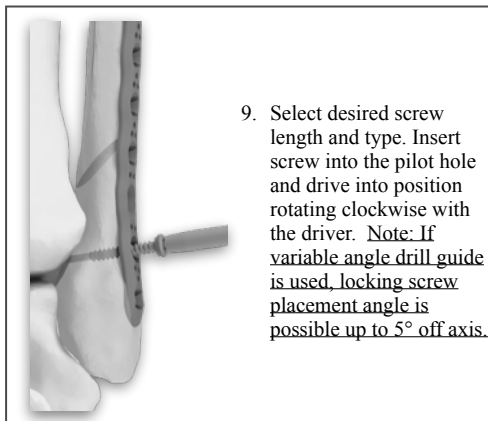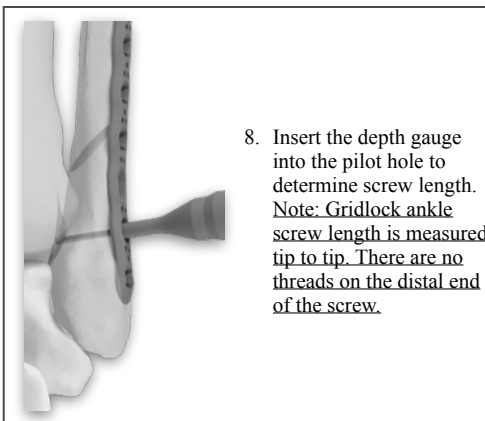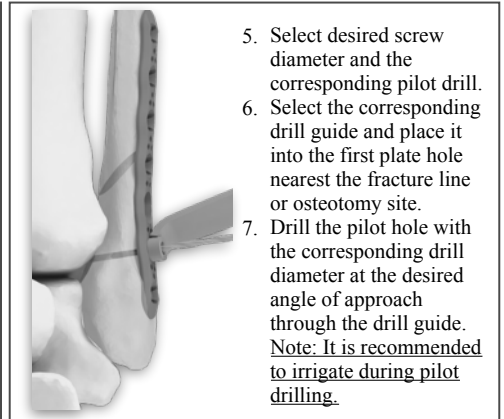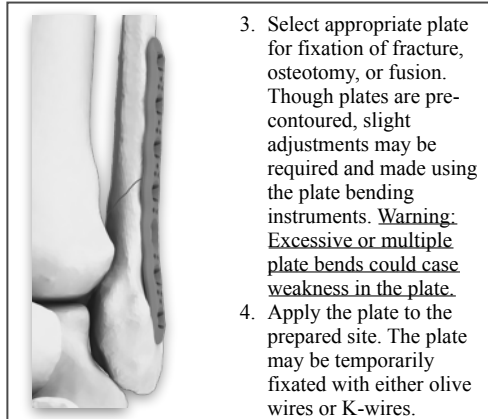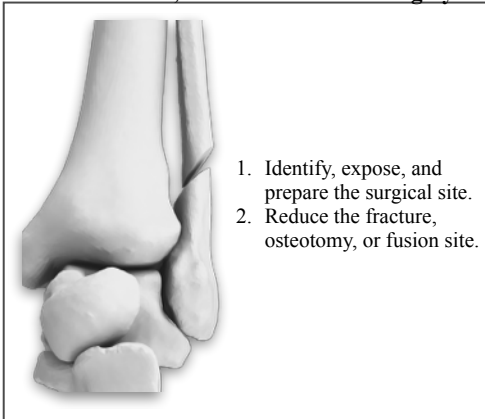

Instructions for use, Syndesmotic Screws (Optional)

1. Once placement of all desired screws in plate is complete, stabilize the syndesmotic joint using the syndesmotic bone clamp.
2. Place the syndesmotic drill guide inside the syndesmotic hole in the plate.
3. Drill the pilot hole with the syndesmotic drill bit. *Note: It is recommended to irrigate during drilling.*

4. Insert the Gridlock Ankle depth gauge in the pilot hole to determine screw length.
5. Using a handle and the ankle screw driver, drive the syndesmotic screw into the pilot hole rotating clockwise until the desired position is achieved.

Instructions for use, Tiger Cannulated Screws (Optional)



eIFU: www.trilliantsurgical.com/regulatory

1. Reduce fracture, osteotomy, or fusion site.
2. Insert appropriate K-wire to the correct length under image intensification. Avoid bending the K-wire by inserting in 15mm - 20mm increments.
3. Slide the countersink over the guide wire until the countersink tip contacts bone. Rotate the countersink clockwise and counterclockwise to create the necessary recess in bone.
4. Slide the appropriate depth gauge over the K-wire advancing until the depth gauge contacts bone. Measure for the desired screw length by examining the end of the K-wire in relation to the marks on the depth gauge.

5. It is recommended to pre-drill to reduce the axial force necessary for inserting the screw in cases of dense bone.
6. Remove the desired Tiger Cannulated Screw from the screw block. Slide the screw over the K-wire.
7. Using a handle and the appropriate driver, drive the Tiger Cannulated Screw into bone rotating clockwise until the desired fixation is achieved.
8. Remove and discard the guide wire.

Instructions for use, Solid Core Interfrag Screws (Optional)

An interfragmentary screw may be used in conjunction with a plate. If using a solid core interfrag screw, follow the lag technique for solid core fully threaded screws.

1. Reduce fracture, osteotomy, or fusion site.
2. Place the desired diameter drill guide on determined site for screw placement. Drill glide hole in proximal cortex/section of bone.
3. Place the corresponding drill guide inside glide hole and drill pilot hole through distal cortex using the appropriate pilot drill.

4. Countersink the proximal cortex.
5. Using the lag depth gauge, determine the length of the screw required.
6. Select appropriate screw length and insert rotating clockwise with the driver until the desired position is achieved.

Plate Removal (If necessary)

1. Locate plate and screws with intra-operative imaging.
2. Palpate plate and remove surrounding soft tissue to gain maximum exposure.
3. Engage screw heads with appropriate driver and rotate counterclockwise until screws are removed.

4. Use forceps to remove plate.
5. All devices removed should be treated as medical waste and disposed of accordingly.

Cleaning

Non-sterile products must be carefully cleaned prior to sterilization. Trained personnel must perform cleaning and mechanical inspection prior to sterilization. Compliance is required with the equipment manufacturer’s user instructions (manual and/or machine cleaning, ultrasound treatment, etc.) and recommendations for chemical detergents. For validated cleaning instructions, please reference document 900-06-010, Gridlock Ankle Plating System Cleaning and Sterilization Protocol.

Packaging and Sterility

NON-STERILE PRODUCT

The Gridlock Ankle Plating System (instruments and implants) can be packaged non-sterile and therefore must be sterilized prior to surgical use. Use of the sterilizer shall comply with the manufacturer’s user instructions. The user facility must clean and disinfect instruments prior to sterilization per standard hospital procedures. Non-sterile devices are sterilizable by steam sterilization (autoclaving). The following parameters should be followed:

Sterilization Method	Pre-Vacuum Steam	Gravity Steam
Condition	Wrapped*	Wrapped*
Temperature	270°F (132°C)	270°F (132°C)
Time	4 minutes	30 minutes
Dry Time	Recommended 50 minutes**	Recommended 60 minutes**

* The system shall be packaged for sterilization by double wrapping in standard central supply wrap (i.e. Bio-Shield® Sterilization Wrap).

** Trilliant Surgical has validated the recommended sterilization cycle and dry time for trays. The dry time varies due to load configuration, wrapping method, and material.

CAUTION

Federal Law (USA) restricts this device to sale by or on the order of a physician. Do not attempt a surgical procedure with faulty, damaged or suspect Trilliant Surgical instruments or implants. Inspect all components preoperatively to assure utility.

MRI Safety Information

The Gridlock Ankle Plating System has not been evaluated for safety and compatibility in the MR environment. It has not been tested for heating, migration, or image artifact in the MR environment. The safety of the Gridlock Ankle Plating System in the MR environment is unknown. Scanning a patient who has this device may result in patient injury.

Symbol Definitions

- Batch Code Catalogue Number Quantity Do not use if package is damaged Consult instructions for use
- Device only to be sold on or by the order of a physician Manufacturer Date of Manufacture Do not re-use
- Caution Non-Sterile Device has no expiration



Trilliant Surgical LTD
6721 Portwest Drive, Suite 160
Houston, TX 77024
1-800-495-2919

Instructions for Use for all Trilliant Surgical product lines are available for download at www.trilliantsurgical.com/regulatory.
 Please contact company for product inquiries and surgical techniques, or to report any adverse experience.